Pflege- und Reinigungsanleitung
Care and cleaning instructions
Instructions d'entretien et de nettoyage

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Dear valued customer,

by deciding to buy this product from Hermann Bock GmbH, you have opted for a care product that has a long service life and delivers superior functionality at the highest safety level. In order to avoid possible malfunctions and risks for persons, users and third parties, please read these instructions carefully and follow the instructions and notes contained therein.

Sincerely yours

Klaus Bock

Dr. Stefan Kettelhoit

Intended purpose:

- The bedside cabinets and servers are used to support the care of frail and dependent people both in old people's homes and nursing homes as well as in home care field
- The bedside cabinets and servers are not suitable for use in hospitals
- The bedside cabinets and servers are designed for storage, and the support of food intake and facilitate the work of the caregivers
- The bedside cabinets and servers can be positioned and moved on the floor independently from the nursing care bed
- If a table top can be adjusted in height, it can also be used above the lying surface of the nursing care bed
- The bedside cabinets and servers can be used independently by the persons in need of care
- The bedside cabinets and servers may only be operated by instructed persons
- The bedside cabinets and servers are basically suitable for multiple use; however, the precondition for this is the observance of the care and cleaning instructions

Guarantee

Hermann Bock GmbH gives 24 months warranty on the bedside cabinets and servers. If original Bock spare parts are not used, or repairs/changes are made to the bedside cabinets and servers or their components by unauthorised personnel, any warranty shall lapse. Please also observe the notes in the user manuals for the nursing care beds.

If damage should occur despite a proper handling, please contact the manufacturer or your dealer.

Care and cleaning instructions:

The table tops of the bedside cabinets and servers are made of coated decorative chipboard. Depending on the model, a melamine resin coating or an HPL coating (high-pressure laminate) with additional edge protection made of plastic (ABS) is used. The coated table tops are undemanding and do not require any special care because of their resistant and hygienically sealed surface. In general, a routine cleaning is hygienically sufficient when used by the same patient. Disinfection of bedside cabinets and servers is only necessary in case of visible contamination with (potentially) infectious material, or for reprocessing.

In order to maintain the usability of bedside cabinets and servers in the long term, please observe the following care and cleaning instructions:

- The bedside cabinets and servers are not suitable for mechanical cleaning and disinfection, and may only be cleaned and disinfected manually
- Light soiling can be removed with a soft, clean and, if necessary, slightly moistened cloth
- Heavy soiling can be removed with a warm soap / detergent solution or with a commercially available cleaning agent
- Remaining residues (e.g. also oil, wax...) can be removed with organic solvents (e.g. acetone) in most cases
- Only clean soft cloths, sponges and brushes may be used as cleaning aids
- Furniture polishes and cleaning agents containing grease and wax can change the structure of the surface, and may therefore not be used
- Cleaning agents containing strong acids, strong acid salts or abrasive or bleaching components may not be used, as they can damage the material
- Moisture (standing water, dampness) will damage bedside cabinets and servers, especially at the butt joints, and should therefore be avoided at all costs
- High temperatures can damage the table top, so a coaster should always be used for hot pots and cups
- Cutting on laminate can cause scratches and should be avoided
- The bedside cabinets and servers can be disinfected with a wipe disinfectant. Please observe the tested and recognised procedures of the Robert Koch Institute (a list of the recognised commercial cleaning and disinfectants can be found on the Internet at www.rki.de)

For care and cleaning of other products and materials, please observe the general manufacturer's instructions. Additional care and cleaning instructions and relevant information can also be found in the assembly and operation manuals for Hermann Bock nursing care beds.

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Our SALES PARTNERS

Our business partners pursue the same strategy as we do: quality, innovation and above-average standards that are internationally recognized. You can rely on our business partners as you can rely on us.

Please note that only our authorised personnel and our sales partners can provide training, supply of spare parts, repairs, inspections and other service. Otherwise, all warranty claims will be void.

A listing of our current distributors can be found under www.bock.net/kontakt/vertriebspartner

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